

SARS-Related Coronavirus 2 (SARS-CoV-2) Lineage C.37; Lambda Variant \* Isolate: Peru/un-CDC-2-4069945/2021 Culture Fluid (Heat Inactivated) (1 mL) Catalog Number: 0810640CFHI

# PRODUCT DESCRIPTION:

SARS-Related Coronavirus 2 Lineage C.37; Lambda Variant (Isolate: hCoV-19/Peru/un-CDC-2-4069945/2021) is an enveloped, positive-sense single-stranded RNA virus from the Coronaviridae family and the Betacoronaviridae genus. Severe acute respiratory syndrome-related coronavirus 2 (SARS-CoV-2), isolate hCoV-19/Peru/un-CDC-2isolated from a human 4069945/2021 was nasopharyngeal swab in Peru on February 20, 2021. Under the nomenclature system introduced by GISAID (Global Initiative on Sharing All Influenza Data), SARS-CoV-2, isolate hCoV-19/Peru/un-CDC-2-4069945/2021 is assigned lineage C.37 (Pango v.3.1.7 2021-07-09), and GISIAD clade GR using Phylogenetic Assignment of Named Global Outbreak LINeages (PANGOLIN) tool. The complete genome of the clinical isolate of SARS-CoV-2. hCoV-19/Peru/un-CDC-2-4069945/2021 has been sequenced (GISAID: EPI\_ISL\_1591097).

Each frozen aliquot contains 1 mL of heat inactivated viral culture fluid. The pre-inactivation titer was determined from an infectious aliquot.

Viral inactivation is verified after heat inactivation by the absence of viral growth in tissue culture-based infectivity assays.

# **INTENDED USE:**

Heat inactivated viral culture fluids are sold as consumable testing materials. The suitability and performance characteristics should be determined by your laboratory for each intended usage.

These products are NOT intended for use in the manufacture or processing of injectable products subject to licensure under section 351 of the Public Health Service Act or for any other product intended for administration to humans.

FOR RESEARCH USE ONLY. NOT FOR USE IN DIAGNOSTIC PROCEDURES.

### **BIOSAFETY:**

Inactivated SARS-CoV-2 should be used within a Biological Safety Level 2 (BSL-2) facility in a biosafety cabinet (BSC). Please consult your institution's regulations regarding the use of this product. For a detailed discussion on biological safety see the current edition of Biosafety in Microbiological and Biomedical Laboratories (BMBL), published by the CDC.

# PRECAUTIONS:

- Use Universal Precautions, this product is potentially biohazardous.
- Repetitive freezing and thawing is not recommended (aliquot material if necessary).
- To avoid cross-contamination, use separate pipette tips for all reagents.

#### RECOMMENDED STORAGE:

Heat inactivated viral culture fluids should be stored at -65°C or below.

\*The following reagent was deposited by the Centers for Disease Control and Prevention and obtained through BEI Resources, NIAID, NIH: SARS-Related Coronavirus 2, Isolate hCoV-19/Peru/un-CDC-2-4069945/2021 (Lineage C.37; Lambda Variant), NR-55654.

PI0810640CFHI Revision: 02 Effective Date: 07/07/2022

	REF	Catalog Number	X	Temperature Limitation
	LOT	Batch Code	Σ	Expiration Date
	RUO	For Research Use Only	€	Biological Risk
	-	Manufacturer		